Monteringsvejledning (DK)

Montering af R82 neopren vest

Monteringsanvisning (SV)

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Monteringsveiledning (N)

Montering av R82 neopren vest

Mounting instruction (GB)

How to mount the R82 neopren vest

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Montage der R82 neopren Weste

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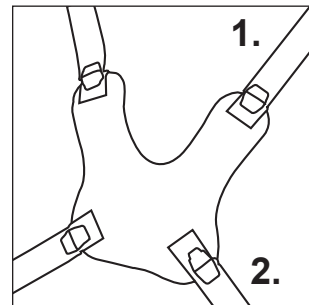
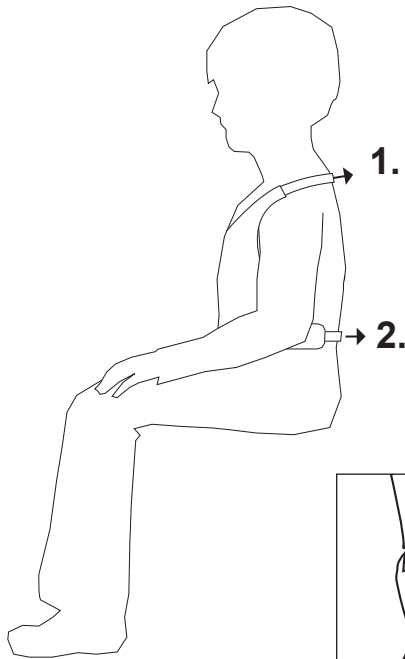
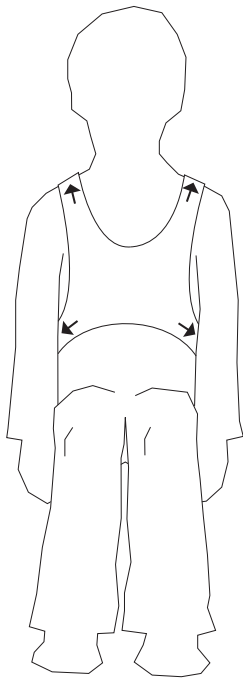
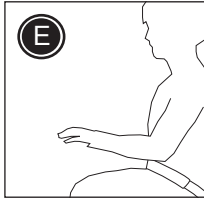
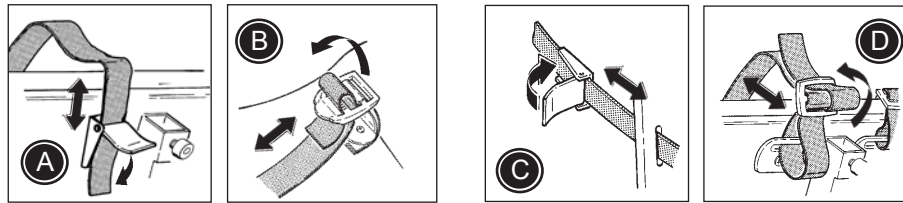
Mounting instruction

How to mount the R82 neopren vest

Recommended for: x:panda, Panda Futura, Panther, Tiger 2000, Serval, Wombat, Winther Reha, Flamingo, Swan

Use the fittings (A) and (B) on the chair to mount the vest. If no fittings are mounted, please use the enclosed fix locks and belt fittings to mount the vest as shown on picture (C) and (D). Any doubt on mounting, please contact a local therapist or authorized R82 personnel.

- * Always mount the shoulder straps (1) a little higher than the shoulder height of the user.
- * Mount the middle straps (2) beneath the user's arms.
- * Mount the lower straps (3) at waist height.

 Always tighten all straps securely to obtain optimal safety, support and comfort. The vest is fitted correctly when two fingers can be put between the harness and the child's body. Check before every use.

 To obtain the best seating position and to prevent the user from sliding down and out of the chair, we recommend always to mount a hip belt (E) together with the vest.

 Never leave the user unattended. Ensure permanent supervision by an adult. Take care that all fixations and adjustments are placed and fitted correctly. Incorrect fitting may cause serious injury. Check before every use.

 Carefully read the care label on the harness to maintain and wash the vest correctly.

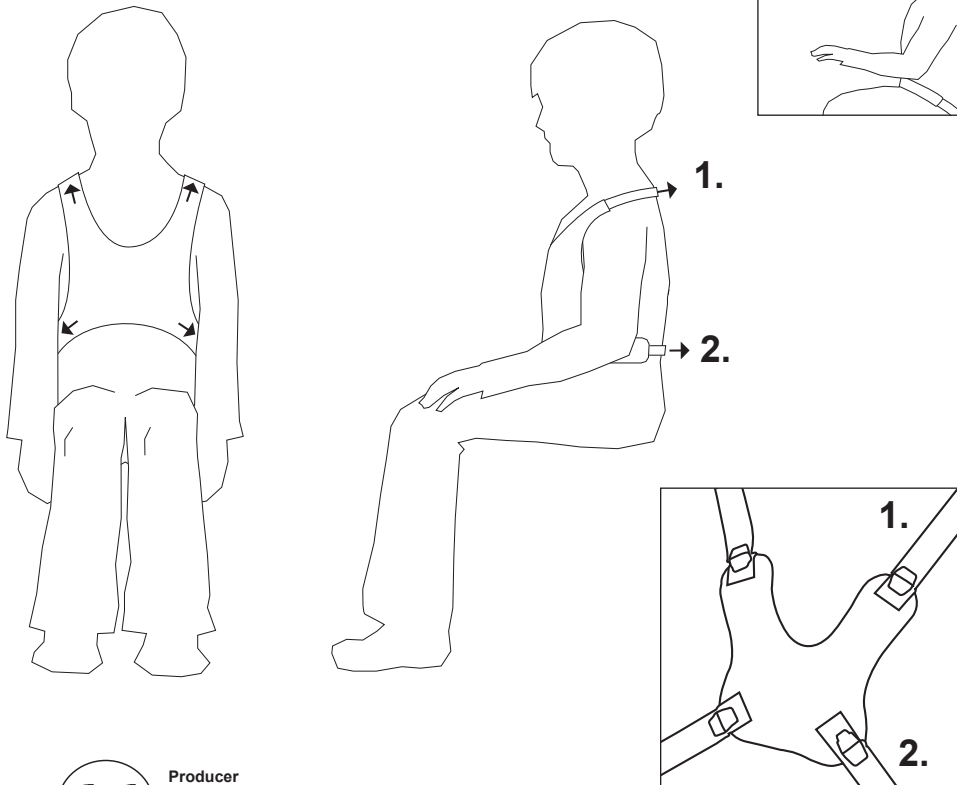
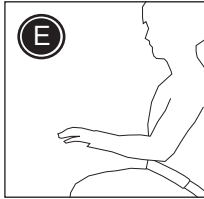
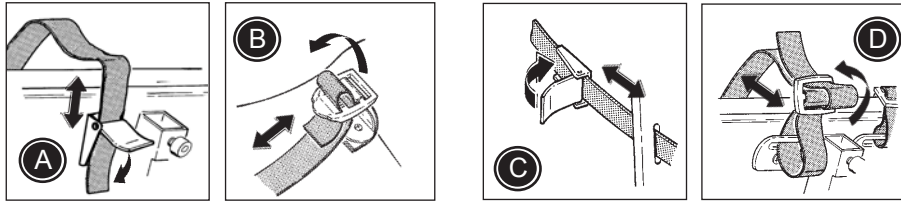
 The latest version of the mounting instruction is always available online: R82.com

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