
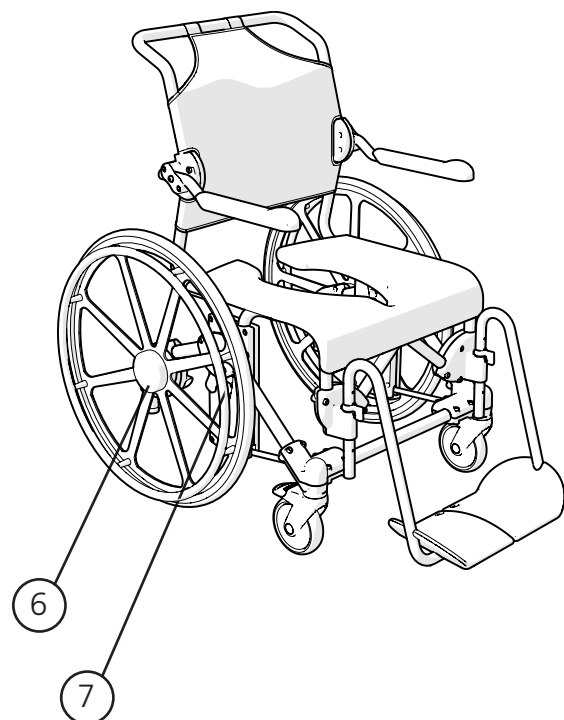
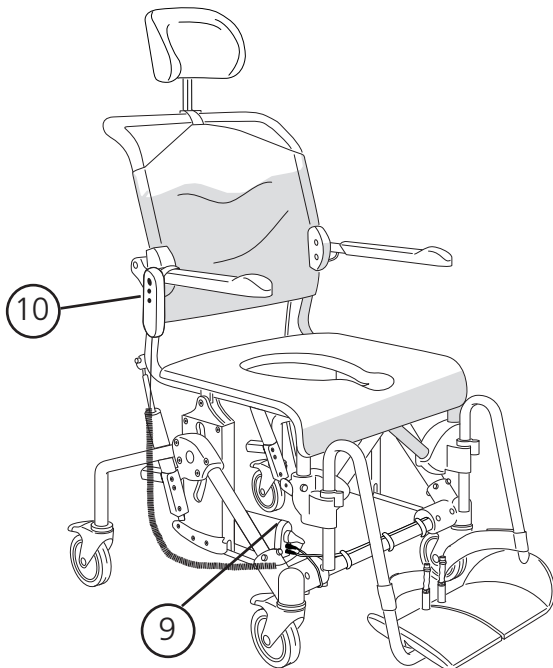
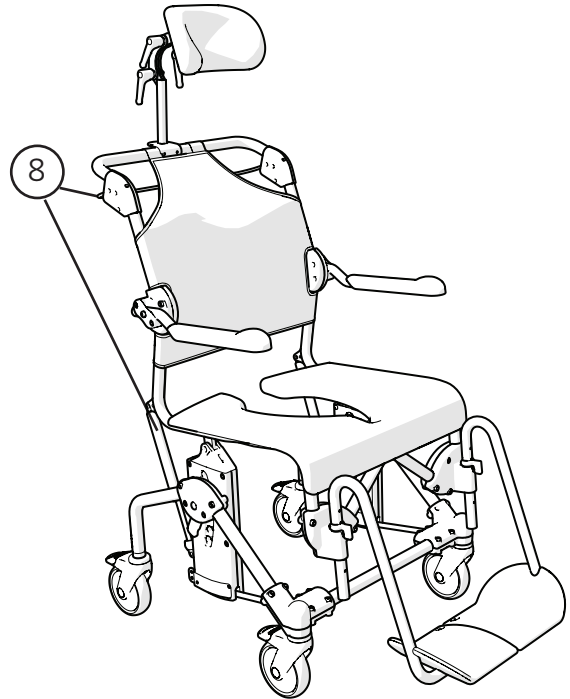
Swift Mobil, Swift Mobil-2 Swift Mobil Tilt, Swift Mobil Tilt-2 Swift Mobil 24", Swift Mobil-2 24"

78320I 20-06-04

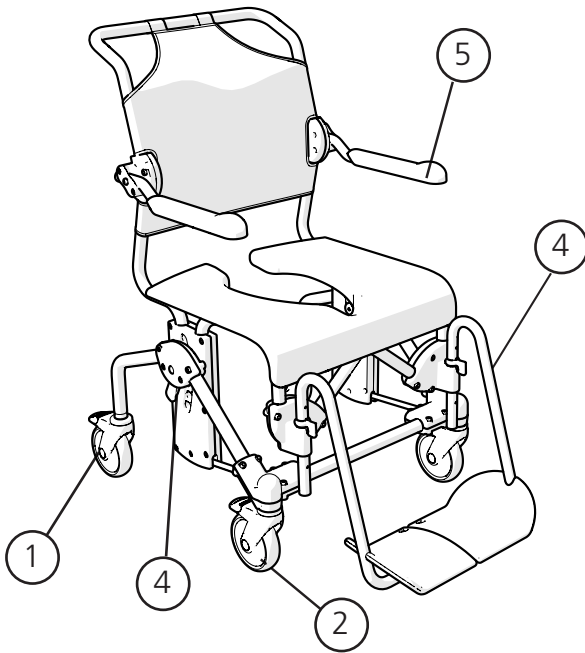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EN

Spare parts can be ordered as per the separate spare parts list for Swift Mobil/Swift Mobil-2. Velcro tapes should be fastened together when washing. The products must be cleaned using conventional non-abrasive cleaning agents. They must be cleaned regularly. The products and their accessories can be autoclaved at 85° for a maximum of 3 minutes and also dried in a drying cabinet at 70° for 10 minutes (unless otherwise stated). Remember that the service life of the product is limited (see manual). This is particularly important when prescribing the product to a user who is expected to use it for an extended period of time.

Check and adjust the following:

- 1) That screws for the castor wheels are properly tightened.
- 2) That the wheels function as intended
 - That the lock function works
 - That the wheels lock satisfactorily. It should not be possible for the wheels to turn in locked position
 - That the chair moves satisfactorily and that the castor wheels rotate
- 3) That the product is assembled correctly
 - That the product and any accessories are assembled correctly according to the assembly instructions
 - That all screws are tightened
- 4) That the chair's adjustment options are working
 - That the locking pin for adjusting the height of the legrests is working and that the legrests can be swung to the side.
 - That the locking mechanism for adjusting the height of the chair is working
- 5) That armrests and armrest attachments are intact
 - That the armrests can be folded up/down
 - Apply pressure to the armrests with your hand in various ways to ensure that the armrests and mountings are intact. Then carry out a functional inspection

Addition for Swift Mobil 24"/ Swift Mobil-2 24"

- 6) That the rear wheel mountings are not loose.
 - If the wheels are loose or have to be changed due to wear:
 - Completely unscrew the wheel and clean the thread with a cloth if necessary
 - Original bolts are treated with "Tufloc" (locking fluid) that should be OK for 1-2 reassemblies. Check the quality of the Tufloc and if necessary drip locking fluid (loctite 243 or equivalent) on the thread according to the manufacturer's instructions and tighten the wheel
 - The rear wheels are properly mounted when they both spin equally freely without play in the mounting
- 7) That the brakes for the rear wheels are in good condition.
 - When the brakes are not applied the brake pin should be located between 18-22 mm from the tyre.
 - When the brakes are applied it should not be possible for the chair to roll backwards with a load of 15 kg on flat ground.

Addition for Swift Mobil Tilt/ Swift MobilTilt-2

- 8) That the tilt function works and that the gas spring lock in the selected position

Addition for Swift Mobil Tilt with motor:

- 9) That motor and batteries are properly mounted and in working order.
 - Check motor- and battery attachments so that nothing is missing or is out of service.
 - Clean contact surfaces on battery and controller with a cloth.
 - Wires, contacts and seal washers must be of Etac sold original products.
- 10) That hand control and charger are in working order.

White paint can be purchased from Etac to touch-up minor chips.
RAL 9003 (gloss 90). Art. no. 84005052

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